

Attendant® 5L Oxygen Concentrator

5-Liter Oxygen Concentrator



#8X899



Once a Year:



Clean cabinet filter;
replace intake filter

Once every 2 years:



Verify the oxygen purity, flow rate
and pressure of your concentrator
by using an oxygen analyzer.

When changing residents:



Discard oxygen tubing, cannula/
mask, oxygen outlet connector, and
humidifier bottle. Replace cabin filter.
Clean concentrator cabinet. Inspect
plugs, cords and components. Check
oxygen concentration.

- Trusted Attendant® brand
- Easy to maintain – just 2 filters are easily accessible and need attention annually
- Low flow rate and oxygen purity indicators standard
- Alarms for power failure, sensor failure and low flow lasting > 10 minutes

Specifications:	
Flow Rate:	0.5 - 5 lpm
O ₂ Purity %:	93% ± 3%*
Outlet Pressure:	5 psi ± 6.23 psi
Power Consumption:	360W (average)
Sound Level:	≤50 db
Dimensions:	21 ³ / ₁₀ "H x 13"W x 10 ¹ / ₁₀ "D
Weight:	36 lbs.
Warranty:	3 years or 12,000 hours of compressor usage

*0.5 - 5 LPM after running for 30 minutes